Equipment trolley
compact-cart OPT Extension
Instruction Manual

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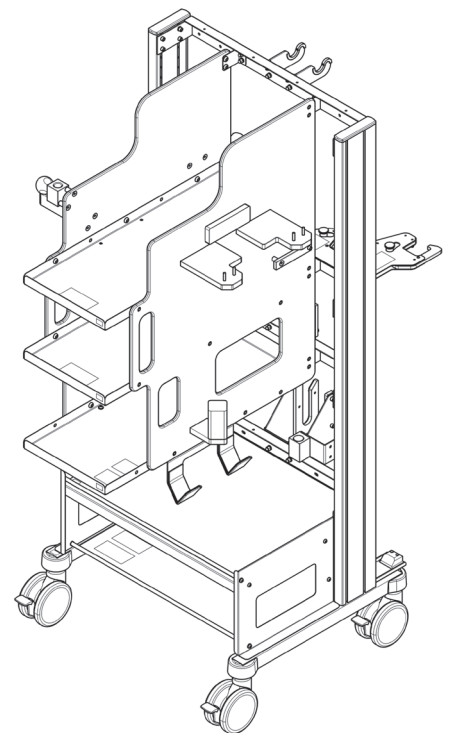
Original Instruction Manual

This is a class I medical device within the meaning of the European Medical Device Regulation(MDR) 2017/745, Appendix VIII.

The manufacturer declares that this product complies with the fundamental requirements pursuant to MDR 2017/745, Appendix IX, as documented by the CE mark.

Manufacturer and Distributing Com- pany	iTD GmbH Jahnstrasse 1 84347 Pfarrkirchen Germany www.itd-cart.com
Sales and Technical Support:	Baxter Medical Systems GmbH + Co. KG Carl-Zeiss-Straße 7-9 07318 Saalfeld Germany
	Telephone +49 3671 586-0
	Fax +49 3671 586-41165 Email: surgical@hillrom.com

We work continuously to develop further.

Please understand that we reserve the right to make changes to the scope of delivery in terms of mode, equipment and technology at any time.

Reprinting, reproduction and translation, even excerpts, is not permitted without the written consent of ITD GmbH. ITD GmbH expressly reserves the copyright in accordance with the law.

The manufacturer is only then responsible for the safety-related features of this device within the scope of the legal provisions if maintenance, servicing and changes are carried out on the device by themselves or a representative according to their instruction.

This instruction manual is valid for the following sales unit:

Description	iTD Material No.	Baxter Material No.
Equipment Trolley compact-cart OPT Extension	KU.5072.800	2067643

Contents

1	Important Information	9
1.1	Proper Use	9
1.2	Improper Use	9
1.3	Symbol Explanation	10
1.4	Safety Information	10
1.4.1	General Information	10
1.4.2	Transport/Storage Change.....	11
1.4.4	Infection Protection	12
1.4.5	Environmental Protection.....	13
1.5	Term Definitions	13
2	Delivery, Unpacking and Setting Up.....	14
3	Overview	15
4	Pictogram on the Product	17
5	Product Combination With Extension Accessories	22
5.1	Extension Adapter and Universal Mount With Hinge	23
5.2	Support Cushion	24
5.3	Base Cushion.....	24
5.4	Counter Traction Bar	25
5.5	Leg Plates	25
5.6	Lower Leg Counter Traction.....	26
5.7	Feed Shaft Unit	26
5.8	Feed Shaft Unit Mount	27
5.9	Double-Joint Strut	27
5.10	X-RAY Extension Strut	28
5.11	MIS Hip Unit	28
5.12	X-RAY Extension Strut	29
5.13	Extraction Bars.....	29
5.14	Knee Crutches by Goepel.....	30
5.15	Standard Block.....	30
5.16	Standard Runner Extension.....	31
5.17	X-RAY Counter Traction Bar and Other Extension Accessories	31
6	Maximum Load	32
7	Handling.....	33
7.1	Put brake on.....	33
7.2	Take brake off.....	34
8	Care, Cleaning and Disinfecting	34

Contents

9 Technical Information35

9.1 Conditions for Operation, Storage and Transport35

9.2 Equipment Trolley35

10 Maintenance and Servicing.....35

1 Important Information

ITD GmbH products are finished for a long and trouble-free service life. Development, construction and production at ITD GmbH have been certified according to DIN EN ISO 13485.

This guarantees:

- highest quality for a long service life,
- simple operation,
- functional design and
- optimised for its purpose.

The products comply with the demands of the German Medical Devices Act (MPG) and bear the CE mark. Please read the instruction manual carefully, from start to finish, before using the product so that you become progressively familiar with the product and the functions available to you. Store the instruction manual near to the product. Use of this equipment trolley compact-cart OPT extension is safe.

At the points concerned the document indicates the residual dangers. Please observe these indications.

1.1 Proper Use

The product described is exclusively determined for human medicine purposes.

The compact-cart OPT Extension equipment trolley is used to store extension accessories produced by Baxter Medical Systems GmbH + Co. KG in an organized manner and transport them ergonomically.

1.2 Improper Use

The permissible load of the equipment trolley compact-cart OPT extension of 150kg must not be exceeded.

The equipment trolley must only be used/operated by qualified staff in a responsible and controlled manner.

These user instructions must be maintained for proper use. Any other use of the equipment trolley is not deemed as proper use.

The supplier/manufacturer will not be held liable for damage to people or property as a result of improper use or operation.

1.3 Symbol Explanation

Important information is identified in this instruction manual with symbols and signal words. Signal words such as **DANGER**, **WARNING**, **CAUTION** and **ATTENTION** indicate the gradation of the danger, symbols highlight this visually. Additional symbols can identify risks of injury or the danger to life. Please pay particular attention to the measures to avert danger!

DANGER

Danger to life (for example electric shock)

Identification of a direct threatening danger, which would result in death or serious injury if the corresponding precautions are not met.

WARNING

Danger to life

Identification of a direct threatening danger, which could result in death or serious injury if the corresponding precautions are not met.

CAUTION

Risk of injury (for example risk of crushing)

Identification of a potential danger, which could result in a moderate to minor injury or damage to property if the corresponding precautions are not met.

ATTENTION

Danger of damage to property

Identification of a potential danger, which could result in damage to property if the corresponding precautions are not met.

1.4 Safety Information

Safety information is measures for the protection of the user and patients from damage that could arise as a result of the use of the equipment trolley.

1.4.1 General Information

- **Risk of tipping.** Do not overload the equipment trolley. Only store the extension parts according to the pictogram on the trolley equipment. The trolley equipment's permitted total weight of 150 kg cannot be exceeded. Observe the load details on the console, the compartments and the lower carrier (base plate) of the equipment trolley.

- **Risk of entrapment and crushing for hands and fingers.** Do not put your hands between the holder and the extension part when placing the extension part of the equipment trolley.
Prior to using the equipment trolley and the other used products, familiarise yourself with their operating principles.
- Only use the equipment trolley when in a braked condition. Release the brake to move the equipment trolley.
- **Danger of personal injury through improper fixing.** Lock the MIS hip unit, double-joint strut and X-RAY extension strut to the equipment trolley. Check the extension accessories for secure hold on the equipment trolley. It is possible that loosened elements may suffer independent movement or slip out.
- Check all mechanical functions and parts prior to commissioning for functionality and integrity. The use of defective or damaged products is prohibited.
- **Risk of injury** by protruding parts.
- Pay attention to the pictogram on the product. All pictograms (stickers) on the product must be complete and legible. Replace damaged or removed stickers.
- The handles are released like a normal screw by turning them anti-clockwise **and tightened by turning clockwise.**

1.4.2 Transport/Storage Change

- **Transportation of people on the equipment trolley is prohibited. Do not sit or stand on the equipment trolley.**
- Occupational safety for staff: Pay attention to correct positioning, particularly of the back, when lifting or carrying the extension accessory because of its heavy weight. If required, pull in another person.
- **Danger of collision with people or objects.** Particular caution should be applied when moving the equipment trolley. Exclude collisions with adjacent people or furniture and impacts.
Secure the movable extension accessory from unforeseeable movement/swinging.
- Release the brakes prior to relocating the equipment trolley. To move the equipment trolley only hold onto the upright strut or handles. Do not steer or guide the trolley by the extension parts, only do this using the handles. To move the equipment trolley, pull in another person. Only move the equipment trolley when the brake is not on.
- **Danger of personal injury through uncontrolled movement.** Some extension parts are only secured into place on the equipment trolley. Independent movement or sliding of the extension parts when moving the equipment trolley is possible. Secure extension accessories that have been put down against sliding.

- **Equipment trolley's risk of tipping due to incorrect loading.** An equipment trolley must only be moved if it has been loaded evenly.

1.4.3 Maintenance/Serviceing

The uni-cart, vexio-cart, pro-cart, duo-cart, classic-cart, compact-cart and endo-cart equipment trolleys have been developed and built for many years of trouble-free use. Check the functionality of the following parts every **12 months** in order to ensure the safety:

Shelves:

- Check that the fastening screws are tight and thereby the shelving is robust and straight.

Rollers:

- Check that the rollers run free and that all of the brakes work.
- Check that the 4 bolts that hold the rollers at the bottom of the base, as well as the rollers themselves, are sitting firmly in the brackets.
- For connecting rollers, the tread must be free from contamination so that its function can be guaranteed.

Serial numbers:

- Compare the serial numbers of the trolley with the information contained in the maintenance book.
- Should you discover a problem when carrying out this check, please notify the supplier immediately.
- Staff (hospital and service staff), who work directly or indirectly with a mobile equipment trolley must be trained.
- Adjustment works can only be carried out by professional staff.
- Repairs and maintenance work can only be carried out by Co. ITD GmbH.
- ITD GmbH is not liable for damage, regardless of the type, as a result of deferred inspections, inadequate maintenance or servicing, as well as changes made to the product.

1.4.4 Infection Protection

- Adhere to all specifications for cleaning/disinfecting. The cleaning and disinfecting procedures and agents described within this instruction manual must be observed.
- Only provide a service engineer with cleaned and disinfected devices and equipment for maintenance and repair works.

1.4.5 Environmental Protection

- Dispose of all cleaning and disinfecting agent remains or residue in a harmless and environmentally-friendly manner.

- Please pay attention to this document for environmentally-friendly disposal.

1.5 Term Definitions

Numbers in square brackets [51]	Number code in graphics (Allocation of graphic - text)
------------------------------------	---

The following terms or abbreviations are used within this instruction manual:

Term	Definition
Extension unit	The extension unit includes the extension adapter and all accessory items that are fixed to it or are required for storage of the extension.
Extension strut	• Double-joint strut with telescopic spar • X-RAY extension strut
Equipment trolley	Equipment trolley compact-cart OPT extension
Baxter	Baxter Medical Systems GmbH + Co. KG
Manufacturer	iTD GmbH

NOTE

Presentation of the basic frame can deviate from actual equipment trolley.

2 Delivery, Unpacking and Setting Up

The equipment trolley is delivered upright and fully assembled in a pallet in a box. The extension accessories are not included within the scope of delivery.

Delivery, unpacking and setting up

Packaging dimensions:	
Length	1200 mm
Width	800 mm
Height	2100 mm
Weight:	
Equipment trolley	78 kg
plus the weight of the packaging.	

ATTENTION **Danger of damage to property**

Packaging is not water-tight. Storage with consideration for the notes in the Technical Information (see chapter 9, page 35).

Unpack the equipment trolley in a room with a level floor and sufficient free space.

When unpacking, follow the subsequent sequence:

1. Remove straps.
2. Pull box upwards.
- 3.

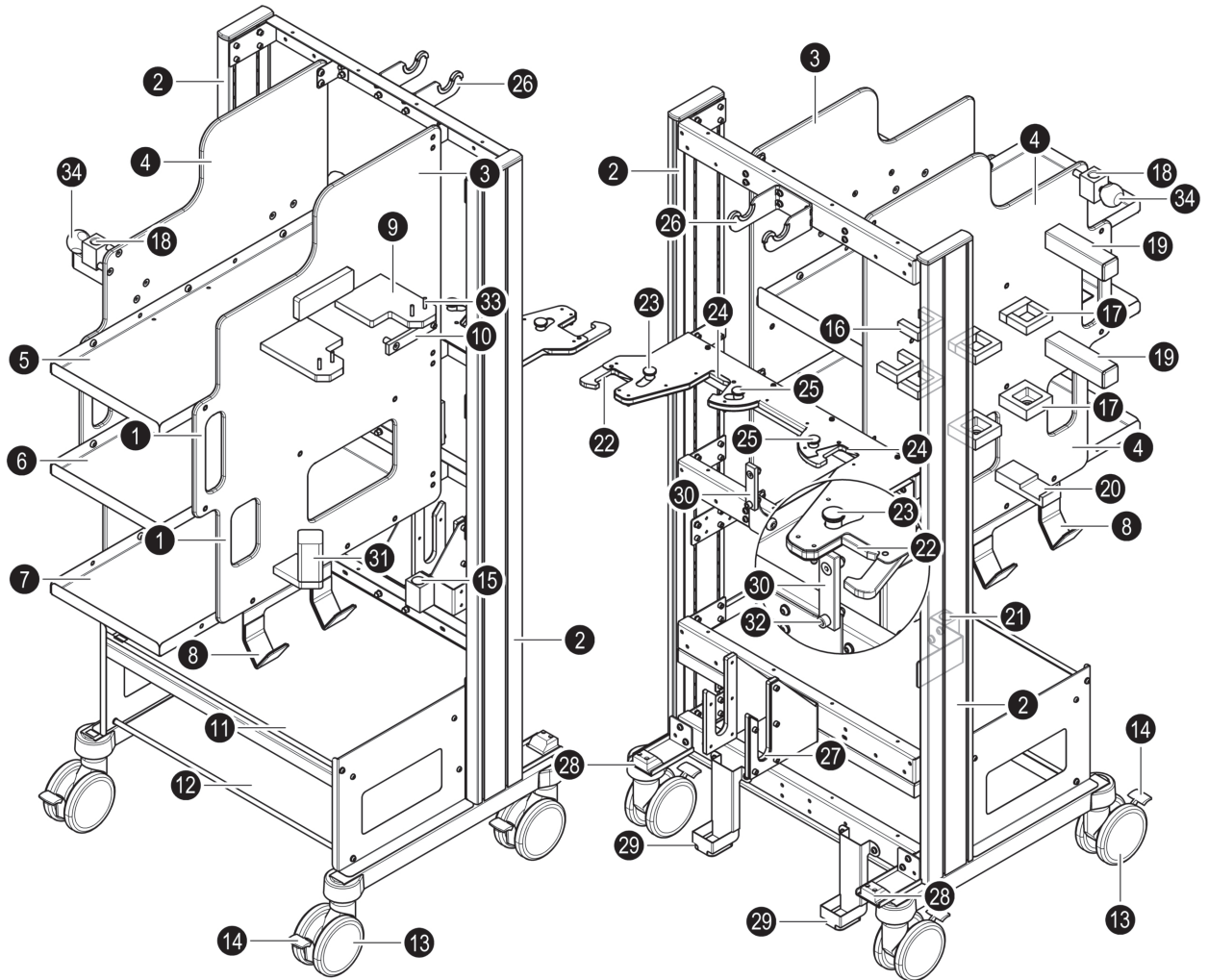
CAUTION **Danger of personal injury. Risk of tipping.**

Lift the equipment trolley from the pallet with at least 2 people.

4. Dispose of the pallet and packaging material in an environmentally-friendly manner.

3 Overview

View from in front (front)

View from behind (back)

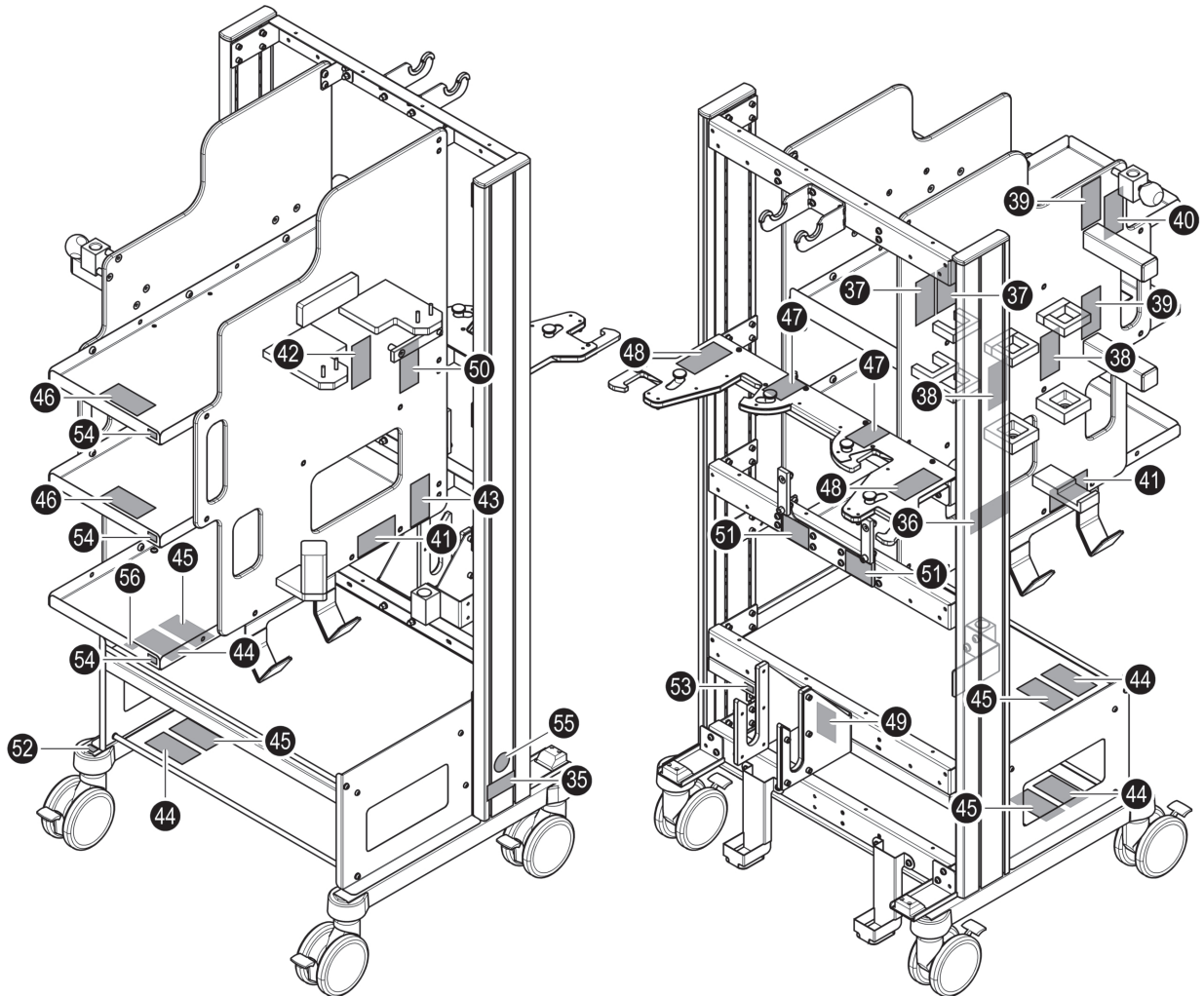
Legend:

- [1] handle for moving the equipment trolley
- [2] strut
- [3] right plastic plate
- [4] left plastic plate
- [5] upper compartment
- [6] middle compartment
- [7] lower compartment
- [8] hook for feed shaft unit
- [9] extension adapter holder
- [10] standard runner for a standard block
- [11] console
- [12] lower carrier (base plate)

Legend:

- [13] twin castor
- [14] parking brake
- [15] mount for counter traction bar
- [16] holder for feed shaft unit mount
- [17] holder for extraction bar
- [18] mount for accessories with bars
- [19] bar connecting pieces for X-RAY extension strut adapter
- [20] holder for lower leg countermove
- [21] insertion opening for the lower leg countermove
- [22] holder for double-joint strut
- [23] operating lever to lock and unlock
- [24] holder for X-RAY extension strut adapter
- [25] operating lever to lock and unlock
- [26] upper holder for MIS hip unit
- [27] lower holder for MIS hip unit
- [28] bar guide for double-joint strut
- [29] bar guide for X-RAY extension strut adapter
- [30] standard rail for standard runner extension
- [31] universal mount holder
- [32] end stop
- [33] extension adapter bolts
- [34] locking handle

4 Pictogram on the product





You will find the following pictograms on the equipment trolley:







View from in front (front)

View from behind (back)





Item	Pictogram	Meaning	Position
[35]		type plate	on the right support profile, bottom
[36]		mount for lower leg counter-move	left plastic plate

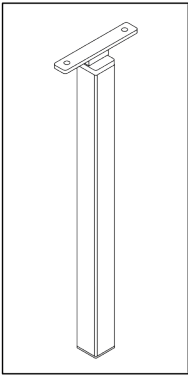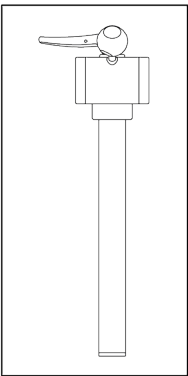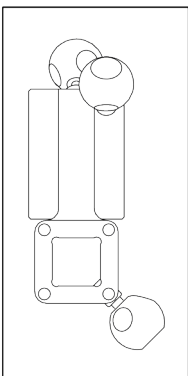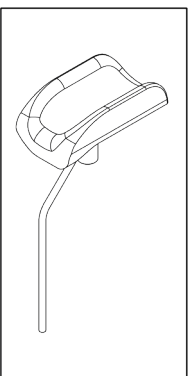Pictogram on the product

Item	Pictogram	Meaning	Position
[37]		holder for feed shaft unit mount	left plastic plate
[38]		holder for extraction bar	left plastic plate
[39]		bar connecting pieces for X-RAY extension strut adapter	left plastic plate
[40]		intake for accessories with bars	left plastic plate

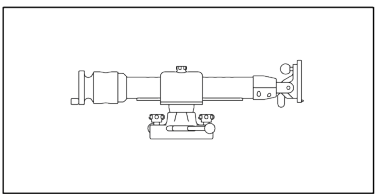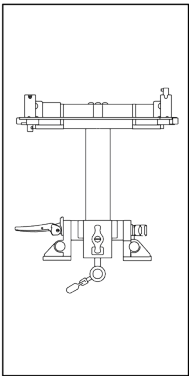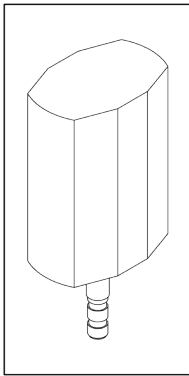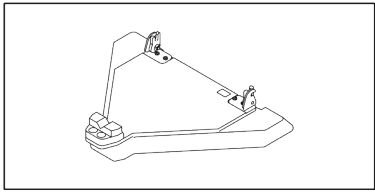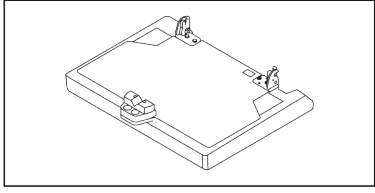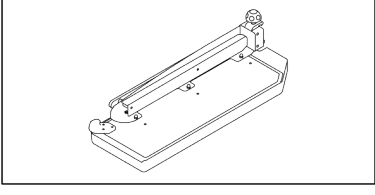Item	Pictogram	Meaning	Position
[41]		hook for feed shaft unit	left and right plastic plate, below
[42]		extension adapter holder and universal mount with hinge	right plastic plate
[43]		intake for counter traction bar	right plastic plate
[44]		storage for support cushion universal	console
[45]		storage for support cushion hip	lower carrier (base plate)
[46]		storage for leg plate	upper and middle storage compartment

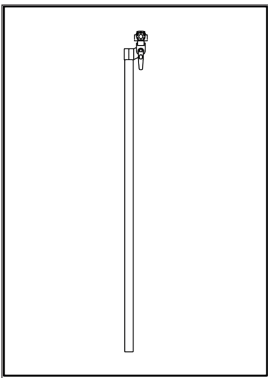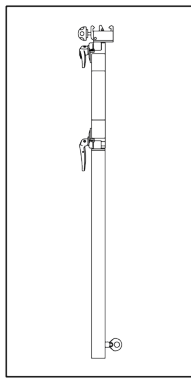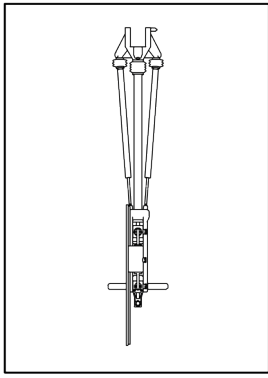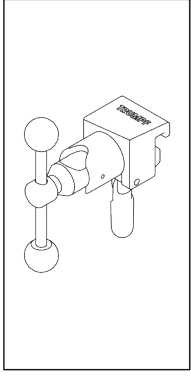Pictogram on the product

Item	Pictogram	Meaning	Position
[47]		holder for X-RAY extension strut	rear-side
[48]		mount for double-joint strut	on the right and left support profile, back
[49]		holder for MIS hip unit	rear-side
[50]		standard runner for standard block	right plastic plate

Item	Pictogram	Meaning	Position
[51]		standard rail for standard runner extension	rear-side
[52]		the permitted total load of the equipment trolley is 150 kg	lower carrier (base plate), right side
[53]		the console can be loaded with a maximum of 50 kg.	rear-side, on the console
[54]		the compartment can be loaded with a maximum of 20 kg.	upper compartment, middle compartment, lower compartment
[55]		follow the Instruction manual	on the right support profile, bottom
[56]		the compartment can be loaded with a maximum of 10 kg.	upper compartment

Type plate:

Legend:

- [A] Manufacturer
- [B] Product name
- [C] Material number
- [D] Serial number
- [E] The device conforms to MDR 2017/745, Annex IX
- [F] Date of manufacture

5 Product Combination With Extension Accessories

The use of the equipment trolley compact-cart OPT extension is permitted in conjunction with the following offered and sold products by the company Baxter Medical Systems. More detailed information about the Baxter extension accessories can be found in the relevant Baxter user manuals.

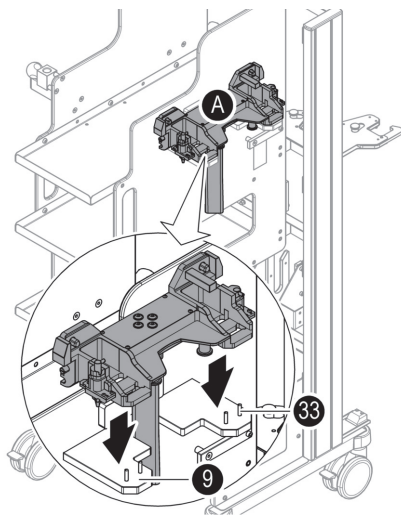
ATTENTION

Danger of personal injury and property damage

Apply the brake to the equipment trolley prior to putting down extension accessories (see page 33).
Only store the individual extension parts according to the pictograms on the equipment trolley.

5.1 Extension Adapter and Universal Mount With Hinge


CAUTION

Risk of entrapment and crushing for hands and fingers


Do not put your hands between the holder and the extension part when placing the extension part of the equipment trolley.

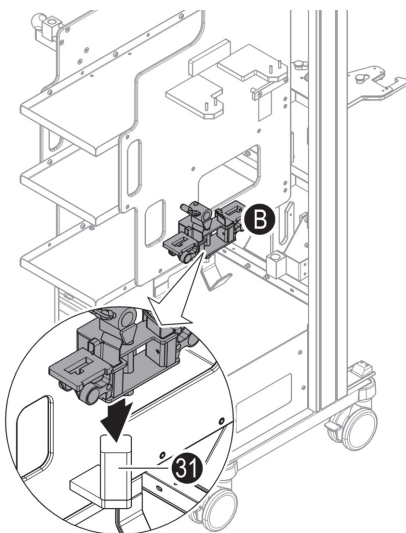Store the extension adapter [A] in the holder [9] on the right plastic plate. Check the extension adapter for secure mounting on the equipment trolley and for correct positioning on the stoppers [33].

Store the universal mount with hinge [B] in the holder [31] on the right plastic plate.

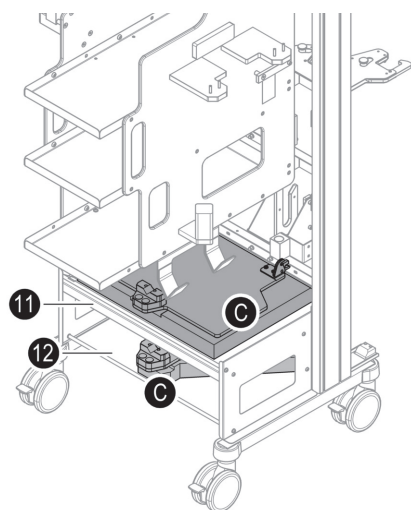
5.2 Support Cushion

Put the support cushion [C] down on the console [11] or on the lower compartment [12] (base plate).


The support cushions must only be secured into place on the equipment trolley.

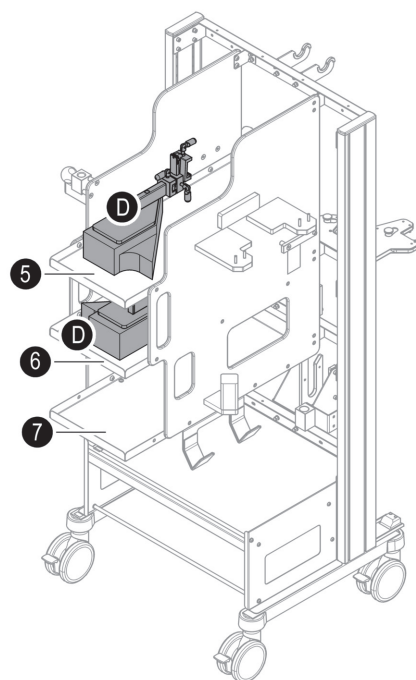5.3 Base Cushion

Base cushions [D] are placed on the compartment ([5], [6] or [7]).

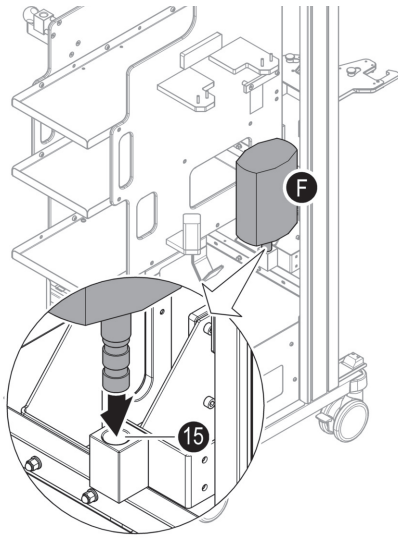
5.4 Counter Traction Bar

Place the counter traction bar¹ [F] with the pivot in the square [15] on the right strut.

Check the counter traction bar for secure hold on the equipment trolley.

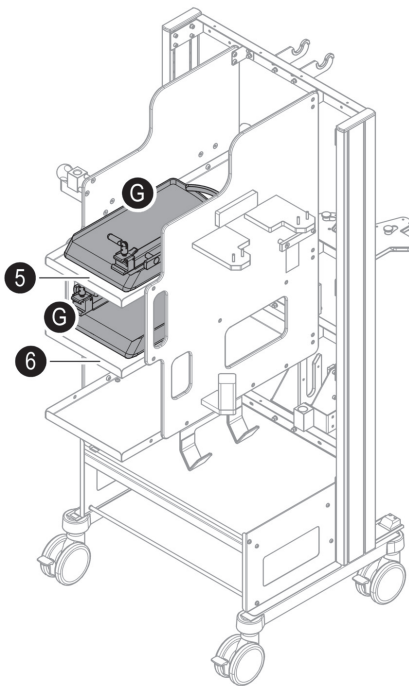5.5 Leg Plates

ATTENTION
Danger of damage to property

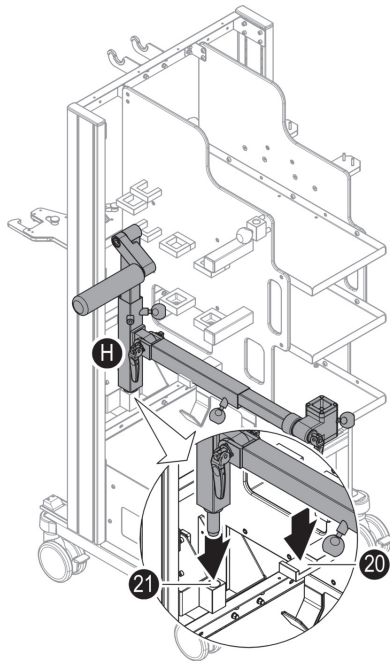
Only place one leg plate on each tray. Do not store any other objects under or on top of the leg plate.

Put the leg plates [G] down in the upper [5] and middle [6] compartment.

The leg plates must only be secured on the equipment trolley.

¹ all versions other than X-RAY counter traction bar

5.6 Lower Leg Counter Traction

CAUTION

Risk of entrapment and crushing for hands and fingers

Do not put your hands between the holder and the extension part when placing the extension part of the equipment trolley.

Store the lower leg counter traction in the holder on the left plastic plate. It is possible to store a completely mounted lower leg counter traction with counter traction bar and tilting adapter.

Putting down

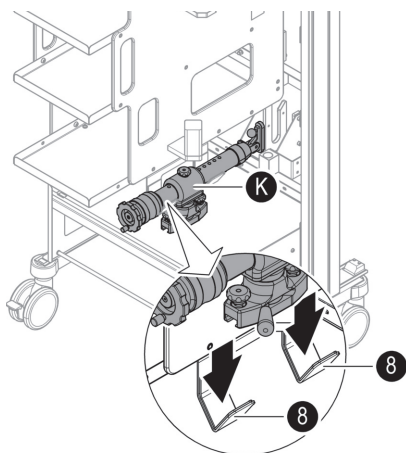
1. Prepare the lower leg counter traction for storage on the equipment trolley.
 - position joint strut vertical to the T-adapter
 - lock the joint to the bar

Description is to be taken from the lower leg counter traction in the instruction manual.
2. Place the lower leg counter traction [H] with the pivot into the insertion opening [21] on the equipment trolley and put it down onto the holder [20] with the joint strut.
3. Check the lower leg counter traction for a secure mount on the equipment trolley.

Removal:

Remove the lower leg counter traction upwards.

5.7 Feed Shaft Unit


CAUTION

Risk of entrapment and crushing for hands and fingers

Do not put your hands between the holder and the extension part when placing the extension part of the equipment trolley.

Deposit the feed shaft unit [K] in the hook [8] below the right or left plastic plate.

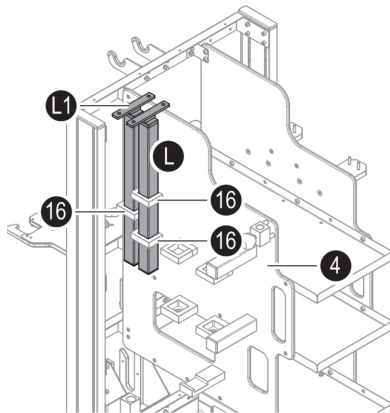
5.8 Feed Shaft Unit Mount

Place the feed shaft unit mount [L] in the holder [16] on the left plastic plate and deposit it with the mount rail onto the plastic plate. The mount rail [L1] must be positioned transverse to the plastic plate [4].

Check the mount for secure hold on the equipment trolley.

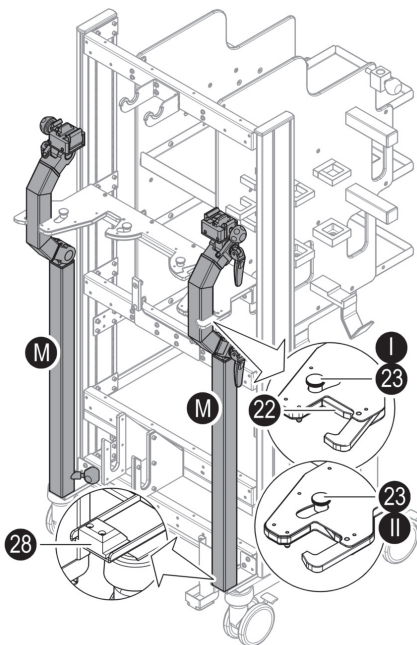5.9 Double-Joint Strut

Store the double-joint strut in the holder on the rear-side of the equipment trolley.

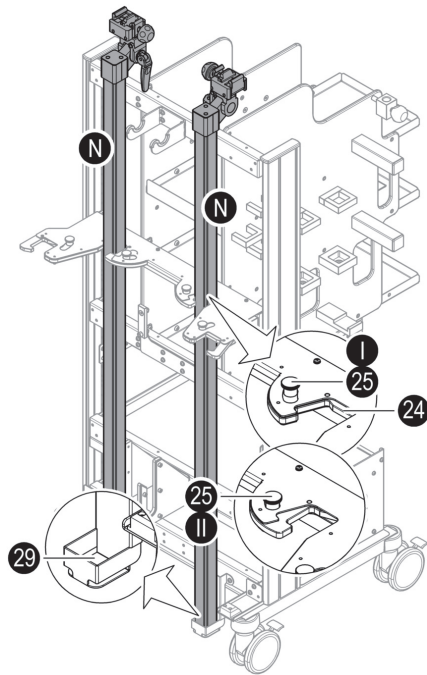
Put down:

1. Lock both joints on the double-joint strut. Description is to be taken from the double-joint strut in the instruction manual.
2. Open the locks on the equipment trolley. To do this, push the operating lever [23] into position [I].
3. Place the double-joint strut [M] with the insertion opening for the extraction bar on the lower bar guide [28]. The crank must point away from the equipment trolley.
4. Press the double-joint strut into the upper holder [22]. The operating lever [23] must audibly lock into position [II].
5. Check the double-joint strut for secure hold on the equipment trolley.

Removal:

1. Securely hold the double-joint strut and open the lock. To do this, push the operating lever [26] into position [I].
2. Tilt the double-joint strut slightly and remove upwards.

5.10 X-RAY Extension Strut

Store the X-RAY extension strut in the holder on the rear-side of the equipment trolley.

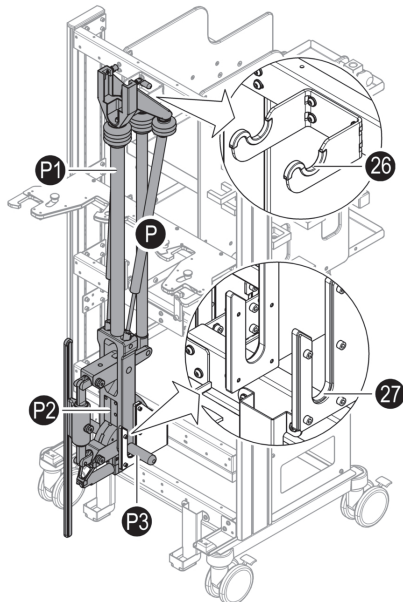
Put down:

1. Lock the joint to the extension strut. Description is to be taken from the extension strut in the instruction manual.
2. Open the lock on the equipment trolley. To do this, push the operating lever [25] into position [I].
3. Place the extension strut [N] in the lower bar guide [29]. The block must point upwards.
4. Press the extension strut into the upper holder [24]. The operating lever [25] must audibly lock into position [II].
5. Check the extension strut for secure hold on the equipment trolley.

Removal:

1. Securely hold the extension strut and open the lock. To do this, push the operating lever [25] into position [I].
2. Tilt the extension strut slightly and remove upwards.

5.11 MIS Hip Unit

Store the MIS hip unit in the holder on the rear-side of the equipment trolley.

1.

CAUTION

Risk of entrapment and crushing for hands and fingers

Do not put your hands between the holder and the extension part when placing the extension part of the equipment trolley. If required, pull in another person.

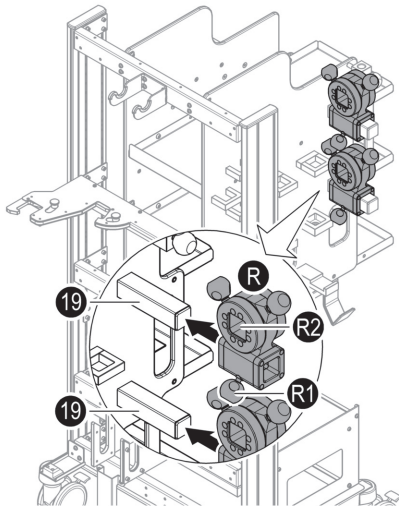
Securely hold the MC hip unit [P] on the traction bar [P1] and on the swivel unit [P2].

2. Deposit the MIS hip unit [P] with the handles [P3] of the swivel unit [P2] in the lower holder [27] of the equipment trolley and with the restraining bar (base holder) in the upper holder [26]. The mounting rail must point away from the equipment trolley.
3. Check the MIS hip unit for secure hold on the equipment trolley.

Removal:

Securely hold MIS hip unit and remove upwards out of the holder.

5.12 X-RAY Extension Strut

Store the X-RAY extension strut adapter on the left plastic plate.

Assembly:

1. Undo locking handle [R1] on the adapter (square tube) [R].
2. Push the adapter [R] with the insertion opening until it touches the strut connecting pieces [19]. The insertion opening [R2] for the X-RAY extension strut must point upwards. Ensure that the adapter is sitting entirely on the strut connecting piece.
3. Close the locking handle [R1] on the adapter.
4. Check the adapter for secure hold on the equipment trolley.

Removal:

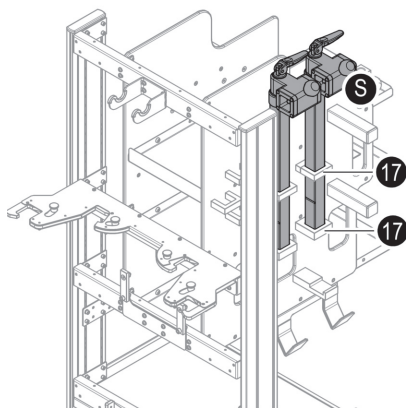
NOTE

Do not completely unscrew the locking handle. Individual pieces can be quickly lost.

Undo the locking handle [R1] on the adapter (square tube) and pull the adapter from the connecting pieces on the equipment trolley.


5.13 Extraction Bars

Push the extraction bar [S] through the upper holder [17] on the left plastic plate and move it onto the lower holder [17].

Check the extraction bars for secure hold on the equipment trolley.

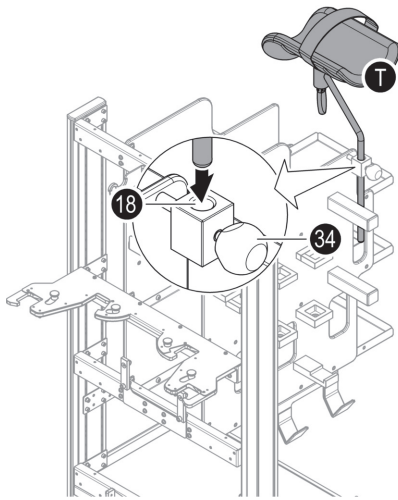5.14 Knee Crutches by Goepel

Store knee crutches [T] in the mount [18] on the left plastic plate.

Assembly:

1. Undo the locking handle [34] on the mount [18].
2. Place the knee crutch rod [T] into the mount opening [18] on the equipment trolley.
3. Position the knee crutch [T] and do up the locking handle [34].
4. Check the knee crutch for secure hold on the equipment trolley.

Removal:

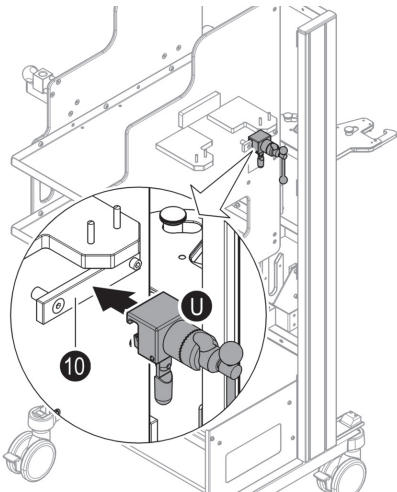
CAUTION
Risk of injury through slipping down

When the locking handle is loosened, the inserter knee crutch slips downwards. Securely hold the knee crutch when removing.

Unscrew the locking handle and remove the knee crutch upwards.

5.15 Standard Block


ATTENTION
Risk of damage to property on the equipment trolley

Only store blocks without accessories on the equipment trolley.

Mount and fix blocks [U] to the standard runner [10] on the right plastic plate.


To couple/decouple also observe the block's instruction manual.

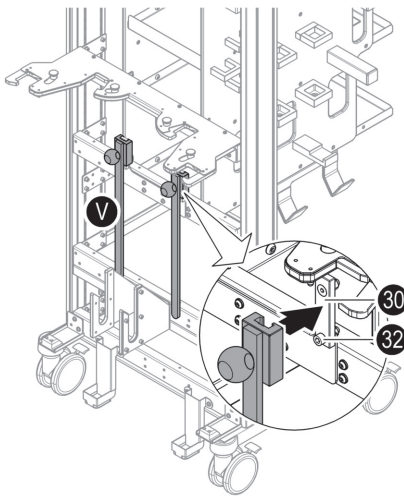5.16 Standard Runner Extension

Push the standard runner extension [V] with the block from above onto the standard runner [30] on the rear of the equipment trolley. The screw [32] prevent the standard runner extension [V] from slipping from the standard runner [30] and falling down.

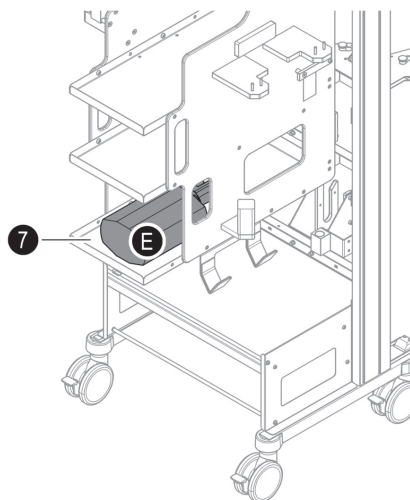5.17 X-RAY Counter Traction Bar and Other Extension Accessories

Deposit the X-ray counter traction bar [E] and other extension accessories on the lower compartment [7].

The extension parts must only be secured into place on the equipment trolley.

6 Maximum Load

**WARNING**
Risk of tipping

The trolley equipment's permitted total weight of 150 kg cannot be exceeded. Observe the load details on the console, the shelf and the lower carrier (base plate) of the equipment trolley.


**CAUTION**
Equipment trolley's risk of tipping due to incorrect loading

Do not overload the equipment trolley. Only store the extension parts according to the pictogram on the trolley equipment.

Description	Load
Total weight of the equipment trolley	210 kg (own weight + additional load)
console	50 kg
storage compartment	20 kg

7

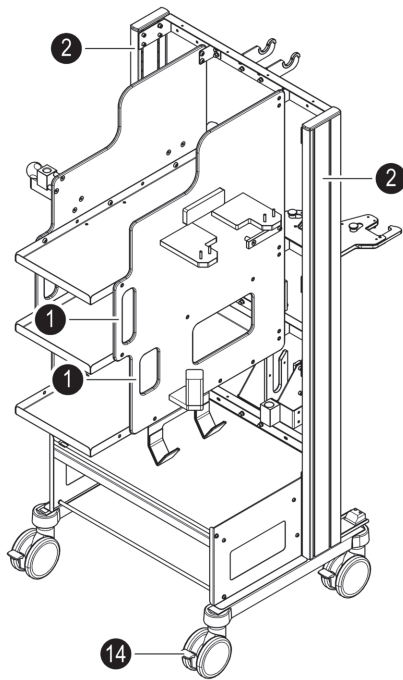Handling

CAUTION
Equipment trolley's risk of tipping

An equipment trolley must only be moved if it has been loaded evenly.
 Only move equipment trolleys on an even floor surface with a maximum gradient of 5°. The equipment trolley's maximum passable elevation is 7 mm.

CAUTION
Risk of collision with people or objects

Particular caution should be applied when moving the equipment trolley. Exclude collisions with adjacent people or furniture and impacts. Secure the movable extension accessory from unforeseeable movement/swinging.

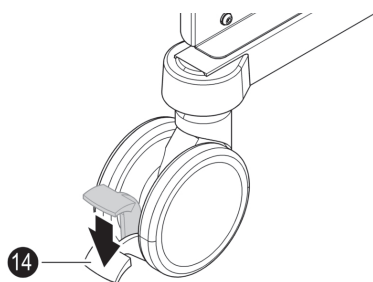
CAUTION
Danger of personal injury due to uncontrolled movement

Some extension parts are only secured into place on the equipment trolley. Independent movement or sliding of the extension parts when moving the equipment trolley is possible. Secure extension accessories that are removed against sliding. Only use the equipment trolley when in a braked condition. Release the brake to move the equipment trolley.

All four wheels have a parking brake [14] and can be blocked. In an unbraked condition, all four wheels have free swivel range and the trolley can be pushed in any direction. Equipment trolleys should only be held on and pushed by the vertical struts [2] or handles [1]. The trolley should not be steered or guided by the extension parts. Where possible, the loaded equipment trolley should be moved by 2 people. Only move the equipment trolley when the brake is not on.


7.1

Put brake on

Move the parking brake [14] on all twin castors downwards. The equipment trolley is braked and cannot roll away.

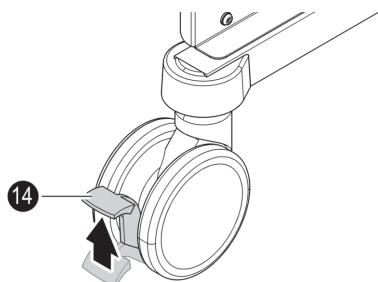7.2 Take brake off

Move the parking brake [14] on all twin castors upwards. The equipment trolley is ready to move.

8 Care, Cleaning and Disinfecting

ATTENTION

Danger of damage to property through incorrect care of the product

The equipment trolley is not machine washable. Do not use a pressure washer. The high pressure of the water jet can push liquid through gaps and joints into the interior and damage the product (for example through corrosion).

Incorrect care products damage the surface.

For maintaining the product, generally do not use any cleaning or disinfecting agents containing abrasive, halide or peracetic acid ingredients. Do not use any cleaning or disinfecting agents containing alcohol or solvents (flammable) on the plastic parts and upholstery.

The cleaning agents for the cleaning and disinfecting must comply with the nationally applicable provision for the medical industry and/or be listed on the DGHM² / VAH³ list. Observe the hospital's hygiene regulations.

Urgently clean dirty products. Cleaning/disinfecting is restricted to regularly wiping down (no immersion) with suitable agents.

For cleaning stainless steel, material parts and painted surfaces, use a pH-neutral or weak alkaline all-purpose cleaner with surfactants as an active cleaning component. For heavy soiling use the cleaner concentrated and subsequently wipe with purified water.

Surface disinfecting agents with an alcohol or aldehyde base are only suitable for the disinfection of stainless steel. For plastic parts and painted surfaces, only use surface disinfectants with an aldehyde base, as the alcohol-containing agents could damage

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the surface. Observe the instructions for use by the disinfectant manufacturer when disinfecting.

After cleaning and disinfecting, remove surplus moisture accumulations, for example drops on the lower edge of the console, with a dry cloth.

9 Technical Information

9.1 Conditions for Operation, Storage and Transport

Temperature	Operation	+10 °C to +40 °C
	Storage/transport	-20 °C to +60 °C
Humidity	Operation	20 % to 80 %
	Storage/transport	5 % to 95 %
Air pressure	min. 700 mbar to 1060 mbar	

9.2 Equipment Trolley

Dimensions (L x W x H)	without extension	1025 mm x 756 mm x 1670 mm
	with extension	1120 mm x 1040 mm x 1885 mm
twin castor	diameter	125 mm
weight:	without extension (own weight)	78 kg
	with extension	210 kg
	permitted total load	150 kg
additional load	console	50 kg
	storage compartment	20 kg

10 Maintenance and Servicing

Repairs can only be carried out by iTD GmbH or a member of staff authorised, trained and certified to do so by iTD GmbH. iTD GmbH is not liable for damage, regardless of the type, as a result of deferred inspections, inadequate maintenance or servicing, as well as changes made to the product.

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